



SG Diagnostics
Committed to Quality of Health



SG Diagnostics COVID-19 Immunity Rapid Test Kit (Anti-Spike RBD IgG Antibody)

The COVID-19 Immunity Rapid Test Kit is an *in vitro* assay for qualitative detection of anti-SARS-CoV-2 Spike RBD IgG from blood sample. It is intended to give an indication of individual's immunity strength after vaccination or past COVID-19 infection.

How it helps you



Confirm immunity from past infection or vaccine



96.5% sensitivity and 98.0% specificity



Rapid detection time of less than 15 minutes



Easy to use by anybody & anywhere



Easy to store at room temperature

COVID-19 Immunity Rapid Test Kit is a *fast* and *accurate* test that reassure your immunity against a COVID-19 infection.

- On-site testing, no lab needed
- East to use with all essential materials provided
- Gives quick results within 15 minutes

Suitable to use at :



Airports



Corporations



Events



Hotels



Homes


Steps for COVID-19 Immunity Rapid Test Kit

1



Use a lancet to do a finger prick. Collect blood sample with the provided pipette.


2

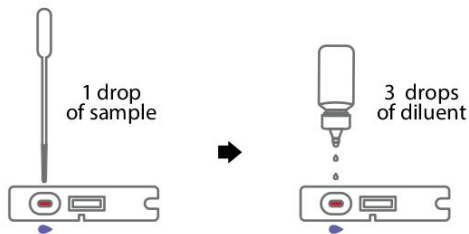

For whole blood / serum / plasma samples, use the provided pipette to add 1 drop of sample (about 20 μ L) to the sample well (●). Add 3 drops of diluent (about 60 μ L) to the sample well.

3

10 - 15
minutes

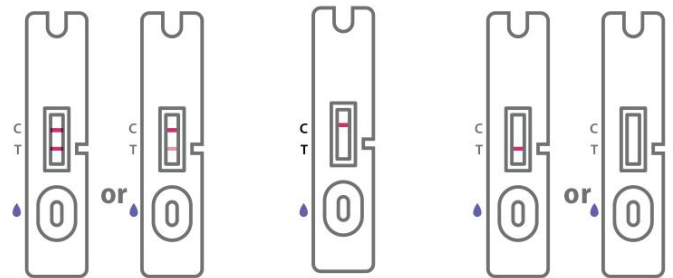

Positive

Negative

Invalid

Wait for the colored band(s) to appear. The result should be read after 10 minutes. Do not interpret the result after 15 minutes. If two red-colored lines mean that you have immunity strength. Single red-colored control line means negative result, thus no immunity. If no line are seen or only test line is seen, the test is invalid and need to be repeated again.

Box Content

- Immunity test cassette
- Diluent
- Pipette
- Instructions for use

Specification

20 tests/kit, 5 tests/kit, 2 tests/kit

Specimen Type

Whole blood, serum or plasma specimen

Total Time

within 15 min

Storage Condition

4°C - 30°C, sealed

Get in touch with us

Germonizer Pte Ltd
39 Woodlands Close, Mega@woodlands
#08-63 Singapore 737856



+65 6747 3631



info@germonizer.com



www.germonizer.com